

Mobilize Rescue Systems™

Implementation Guide

ZOLL®

Welcome to the ZOLL Community

You are just a few steps away from empowering your entire organization to respond to medical emergencies with Mobilize Rescue Systems.

HOW TO INSTALL PUBLIC ACCESS WALL BOXES AND ALARMED WALL CABINETS

Mount Public Access Wall Boxes and Alarmed Wall Cabinets in high-traffic areas with clear visibility and easy access.

Mounting and alarming instructions can be found in the links on this page.

ZOLL Recommends:

- + Mount securely to wall
- + Write licensing code on the Public Access Wall Boxes
- + Set and test alarm wearing ear protection
- + Read [ADA Guidelines](#)



Public Access Wall Boxes

[MODEL 50](#)

[MODEL 100](#)


[HOW TO ALARM](#)



**Comprehensive & Mobile
ALARMED WALL CABINET**

HOW TO REGISTER AND ACTIVATE THE MOBILIZE RESCUE APP IN YOUR UNITS

COMPREHENSIVE AND MOBILE RESCUE SYSTEMS

1. Ensure units are fully charged.
2. Connect to WiFi for the initial activation of each device
 - + WiFi connection is in the settings menu. 
 - + A mobile device hotspot will work if WiFi is not available.
 - + Once a unit is activated, WiFi is not required for use.
3. Step-by-step instructions to activate the Mobilize Rescue App on Comprehensive & Mobile units can be found [HERE](#)
4. Install tamper-proof seals and inspection card
 - + Instructions: [Comprehensive](#) | [Mobile](#)

